

17th April 2019

URGENT: FIELD SAFETY NOTICE – MDS-19-1441

BD/Impromediform GmbH MaxZero™ Needleless Connector and
associated extension sets
All product codes - Refer to Appendix 1
All lot numbers

Attention: Clinical Personnel, Risk Managers, Biomedical Personnel

This letter contains important information which requires your **immediate** attention.

Dear Customer,

BD is issuing an advisory Field Safety Notice related to the **BD MaxZero™ Needleless Connector** (figure 1) and associated extension sets that incorporate the BD MaxZero™ Needleless Connector. The BD MaxZero™ is used with an IV line and/or IV catheter during IV therapy; it can be used to administer IV fluids to the venous system or as secondary sterile injection site for the delivery of fluids to a patient's vascular system through a cannula. The BD MaxZero™ can be used as a standalone device or is supplied as part of an extension set. Appendix 1 contains a full list of impacted product codes. According to our distribution records your organisation may have received these devices.



Figure 1: BD MaxZero™ Needleless Connector

This Field Safety Notice also applies to extension sets manufactured by **Impromediform GmbH** which incorporate the BD MaxZero™ Needleless Connector. The Impromediform impacted catalogue codes are also listed in Appendix 1.

Description of the Problem

Based on customer feedback, BD has identified that there is the potential for droplets of fluid to separate from the sealed surface of the BD MaxZero™ following disconnection. As a result of this, there is the potential for the healthcare professional, the patient or the surrounding area to be exposed to these droplets.

As a result of this feedback and to mitigate the occurrence of droplet separation, BD is updating the Instructions for Use (IFU) with the following recommendation related to disconnecting the device:

“When disconnecting a luer-lock or luer-slip syringe or tubing set from the MaxZero connector, carefully rotate the luer counter clockwise a 360-degree turn using a controlled motion, to minimize fluid escaping”.



Advice on action to be taken:

- 1) Distribute this notice to all those who need to be aware of the update to the disconnection technique of the BD MaxZero™ and the associated extension sets within your organisation.
- 2) Complete the Customer Acknowledgement Form and return it to BD at BDUKFieldAction@bd.com no later than the 29th April 2019.

There is no requirement for customers to return any BD MaxZero™ product to BD. These products can continue to be used in accordance with the guidance in this safety notice & the updated Instructions for Use.

Should you have any questions or require assistance relating to this Field Safety Corrective Action, please contact your Local BD representative at BDUKFieldAction@bd.com

We confirm that the appropriate regulatory agencies have been informed of these actions.

BD is committed to advancing the world of health. Our primary objectives are patient and user safety and providing you with quality products. We apologize for any inconvenience this issue may have caused you and thank you in advance for helping us to resolve this matter as quickly and effectively as possible.

Yours sincerely,

A handwritten signature in blue ink that reads 'William David'.

William David
Senior Director, Quality Compliance EMEA



Appendix 1: Affected Products including extension sets that contain the MaxZero™ Needleless Connector

BD Catalogue Codes			
Product Code	Product Name	Product Code	Product Name
MZ1000	BD MaxZero™ Needle-less Connector	MZ5301	BD MaxZero™. Minibore pressure rated extension set, IV connector
MZ5302	BD MaxZero™. Minibore pressure rated extension set, removable IV connector	MZ5303	BD MaxZero™. Pressure rated extension set, IV connector
MZ5304	BD MaxZero™. Pressure rated extension set, removable IV connector	MZ5305	BD MaxZero™. Pressure rated extension set, IV connector, no clamp
MZ5306	BD MaxZero™. Minibore pressure rated extension set, IV connector, no clamp	MZ5307	BD MaxZero™. Minibore bi-fuse pressure rated extension set, 2 IV connectors
MZ5309	BD MaxZero™. Pressure rated extension set, IV connector	MZ5310	BD MaxZero™. Pressure rated extension set, removable IV connector
MZ9226	BD MaxZero™. Microbore extension set, IV connector, filter	MZ9265	BD MaxZero™. Microbore bi-fuse extension set, 2 IV connectors
MZ9266	BD MaxZero™. Microbore tri-fuse extension set, 3 IV connectors	MZ9267	BD MaxZero™. Microbore extension set, IV connector
MZ9270	BD MaxZero™. Microbore tri-fuse extension set, 3 IV connectors, filter	MZ9271	BD MaxZero™. Microbore bi-fuse extension set, 2 IV connectors, filter
MZ9272	BD MaxZero™. Microbore tri-fuse extension set, 3 IV connectors, filter	MZ9273	BD MaxZero™. Microbore tri-fuse extension set, 3 IV connectors, filter
MZ9274	BD MaxZero™. Microbore tri-fuse extension set, 3 IV connectors, 2 filters	MZ9275	BD MaxZero™. Microbore quad-fuse extension set, 4 IV connectors
MZ9276	BD MaxZero™. Microbore tri-fuse extension set, 3 IV connectors, filter	MZ9277	BD MaxZero™. Microbore extension set, IV connector
MZ9278	BD MaxZero™. Microbore tri-fuse extension set, 3 IV connectors, 3 check valves	MZ9279	BD MaxZero™. Minibore bi-fuse extension set, 2 IV connectors, 2 check valves
MZ9281	BD MaxZero™. Minibore tri-fuse extension set, 3 IV connectors, 3 check valves	MZ9283	BD MaxZero™. Minibore quad-fuse extension set, 4 IV connectors, 4 check valves
MZXT9001	BD MaxZero™. Microbore extension set, IV connector, T-connector	MZXT9003	BD MaxZero™. Minibore tri-fuse extension set, 3 IV connectors, colored clamps, T-connector.
MZXT9004	BD MaxZero™. Minibore extension set, IV connector, T-connector.		
Impromediform Catalogue Codes			
MFXT2602MZ	2-way extension set with 2 MaxZero™ connectors, microbore, 2 clamps	MFXT2603MZ	3-way extension set with 3 MaxZero™ connectors, microbore, 3 clamps
MFXT2605MZ	3-way stopcock with 2 MaxZero™ connectors, microbore extension line, 15 cm	MFXT2606MZ	3-way stopcock with 2 MaxZero™ connectors, macrobore extension line, 10 cm
MFXT2607MZ	3-way stopcock with 2 MaxZero™ connectors, macrobore extension line, 25 cm	MFXT2608MZ	3-way stopcock with 2 MaxZero™ connectors, macrobore extension line, 50 cm
MFXT2609MZ	3-way stopcock with 2 MaxZero™ connectors, macrobore extension line, 100cm	MFXT2610MZ	3 way stopcock standalone with 2 MaxZero™
MFXT2601MZ	Extension set with MaxZero™connector, microbore, 1 clamp		



Field Safety Notice Customer Acknowledgement Form

BD MaxZero™ Needleless Connector and associated extension sets
All product codes - refer to Appendix 1
All lot numbers

Please read in conjunction with Field Safety Notice MDS-19-1441 and return completed form as soon as possible or **no later than the 29th April 2019** to BDUKFieldAction@bd.com

Name of Trust	
Name of Hospital / Facility covered by this response	
Hospital / Facility Address	
Email Address	
Telephone Number	
Name	
Signature	
Date	

Please confirm the following by checking the box:

I have read and understood the contents of this Field Safety Notice.

AND complete one of the following boxes:

I will distribute this Field Safety Notice to all those who need to be made aware.

OR

I confirm that our facility **does not have any** of the affected devices listed in this Field Safety Notice.

Please return your completed Acknowledgement Form to: BDUKFieldAction@bd.com