Nujno varnostno obvestilo

Urgent Field Safety Notice

Address

Sales Division

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Westerwaldstr. 4 · D-56579 Rengsdorf

Kurzz.. Tel.: 99-XXXX/ Fax: 99-XXXX

Email: xxx.xxxx@de.LRmed.com

Date: 29.07.2020

Nujno varnostno obvestilo

Urgent Field Safety Notice

zaradi možnosti pretrganja in nitkanja rentgenske kontrastne nitke v kompresah iz gaze (v sestavi KitPack operacijskih setov)
due to breaking and fraying of the x-ray thread in gauze compresses (in KitPack procedure packs)

Spoštovani,

Dear Sir or Madam,

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Today we would like to inform you about the precautionary product recall of gauze compresses with x-ray thread due to a possible breaking or fraying of the x-ray thread used. The affected gauze compresses with x-ray thread are integrated in KitPack procedure packs.

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Based on feedback from three clinics, it has become known that under tension the x-ray thread in the affected gauze compresses can break and fray at individual points.

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The products and batches affected by this defect were immediately quarantined in our warehouse.

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Theoretically, small pieces of thread could occur when the x-ray thread breaks or frays. This could, for example, lead to inflammation and/or granuloma formation when remaining in the body.

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Since the gauze compresses with x-ray thread are part of the KitPack procedure packs delivered to you, the following measures must be taken in the interest of patient safety:

Prosimo, da ne uporabljate prizadetih kompres iz gaze s RK-kontrastno nitko iz KitPack operacijskih setov in jih pred uporabo v operacijski dvorani zavržete. Pred uporabo drugih komponent preverite, ali so morda onesnažene z deli rentgenske niti.

Please do not use the affected gauze compresses with x-ray thread from the Kit-Pack procedure packs and please discard these products before use in the operating theatre. Before using the other components, please check whether they are contaminated by parts of the x-ray thread.

Da ne bi ogrozili operacijskih posegov in za vzdrževanje nemotene oskrbe pacientov, bomo embalažo KitPack setov na zalogi, označili z naslednjimi dodatnimi informacijami:
In order not to endanger the operation and to maintain direct patient care, we will label the KitPack procedure packs still in stock with the following additional information:

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"Attention: Do not use gauze compresses with x-ray thread. Please discard"

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The other components of the respective Kitpack procedure packs are not affected by this and can be used as usual.

Prizadete izdelke bo seveda mogoče brezplačno zamenjati ali pa pridobiti dobropis.
The products will of course be exchanged free of charge or credited on request.

V svoji organizaciji zagotovite, da so vsi uporabniki navedenih izdelkov in druge osebe, ki jih bomo obveščali, seznanjeni s tem nujnim varnostnim obvestilom in nam vrnite potrjeno priloženo potrdilo.
Please ensure in your organisation that all users of the products listed below and other persons to be informed are aware of this Urgent Field Safety Notice and return the enclosed confirmation to us completed.

V kolikor ste izdelke posredovali tretjim osebam, jim prosimo pošljite kopijo teh podatkov.
If you have passed on the products to third parties, please forward a copy of this information to them.

V naprej se vam zahvaljujemo za sodelovanje in se opravičujemo za morebitne nevšečnosti.
We thank you in advance for your cooperation and apologize for any inconvenience.

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i.V. / On behalf of

29.07.2020
Sales department

XXXYYY
Regional Vigilance Officer

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Attachment:

1) Urgent Field Safety Notice due to breaking and fraying of the x-ray thread in gauze compresses (in KitPack proced)

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Description: Field Safety Notice due to breaking and fraying of the x-ray thread in gauze compresses (in KitPack procedure packs)

Ukrepi:
Corrective Actions:

Please do not use the affected gauze compresses with x-ray thread in the KitPack procedure packs anymore.

Please inform all employees who use the products about this Field Safety Notice and confirm that you are not using the affected gauze compresses with x-ray thread.

The undersigned confirms (please tick as appropriate)

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Pošiljatelj: Lohmann & Rauscher GmbH & Co. KG,