

## Artis systems

Be sure to observe the following information when operating your system.



### Correction for: Cleaning and disinfection



#### Warning


Inappropriate cleaning of the system, e.g. letting fluids enter the interior of system, or using harsh cleaning agents.

**Electrical disturbances, corrosion or surface damages and correspondingly, malfunction or failure of the system**

- ◆ Use only substances for cleaning and disinfection, which are recommended.
- ◆ Do not let cleaning liquids seep into the openings of the system, e.g., air openings, gaps between covers.
- ◆ Observe the following cleaning and disinfection instructions.

### Safety information - cleaning units which have a visible belt

Pay special attention when cleaning units (stands) which are equipped with a visible belt.

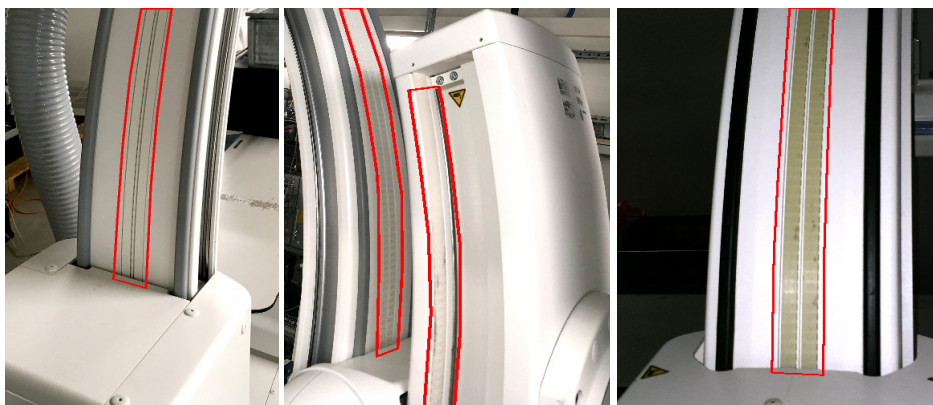

Examples of belts

Observe the following information for cleaning:

- Do *not* use cleaning agents containing *aldehydes*.
- Do *not* use *spray bottles*.
- Perform *dry* cleaning, if possible.
- Use *little* cleaning agent or only water, if possible.
- ◆ *Wipe dry*.