

Positioning the signal input box (SIB/iSIB/ComboBox)

This addendum includes information for all Sensis systems.

Please be sure to observe the following information when operating your system.

This addendum provides information about the positioning of the signal input box (SIB/iSIB/ComboBox).

Throughout this document, the signal input box is referred to as the ComboBox.

WARNING

Incorrect mounting of the ComboBox

Risk of electric shock due to the patient touching the ComboBox or due to spillage of liquids

Risk of system malfunction due to spillage of liquids

Risk of mechanical hazards like falling, vibration, and shaking

- ◆ Mount the ComboBox in a stationary and fixed position.
- ◆ Place the ComboBox out of patient reach.
- ◆ Protect the ComboBox against liquid spillage.

These mounting requirements are fulfilled if Siemens Healthineers positions the ComboBox under the patient table foot end, attached to the dedicated mounting bracket.

For any other mounting positions, additional measures have to be taken to meet these requirements, as, for example, attaching a sterile cover.



If the original mounting position requires modification or if support is needed to achieve proper mounting, contact Siemens Healthineers Customer Service.

Attaching the sterile cover

For liquid spillage protection, sterile covers should be used.

The following should be ensured:

- ✓ The plastic cover of the signal input box is closed.
- ✓ The expiration date on the package of the sterile covers is observed.
- ◆ Attach the sterile cover by pulling it over the housing part facing the ceiling.

If you have any questions about ordering sterile covers, contact Siemens Healthineers.

Positioning the signal input box (SIB/iSIB/ComboBox)

2 / 2

AXA5-200.623.63.02.02

Positioning the signal input box (SIB/iSIB/ComboBox)

Siemens Healthcare Headquarters

Siemens Healthcare GmbH
Henkestr. 127
91052 Erlangen
Germany
Phone: +49 9131 84-0
siemens-healthineers.com

Manufacturer

Siemens Healthcare GmbH
Henkestr. 127
91052 Erlangen
Germany

Original Language: English