

Siemens Healthcare Diagnostics GmbH, SHS EMEA CEET QT, Siemensstrasse 90,
1210 Vienna

Name	M.A. Roland Ertl
Department	SHS EMEA CEET QT
Telephone	+43 51707-38274
Mobile	+43 (664) 8011738274
E-mail	roland.re.ertl@siemens-healthineers.com
Date	March 07, 2022
Document Ref#	POC 22-002.A.OUS

Urgent Field Safety Notice:

Atellica® UAS 800 Urine Sediment Analyzer

Atellica® 1500 Automated Urinalysis System

Device Door Gas Spring Failure

Dear Sirs,

Our records indicate that your facility may have received the following product:

Table 1. Affected Product

Product	Siemens Material Number (SMN)	Software Version
Atellica® UAS 800 Urine Sediment Analyzer Atellica® 1500 Automated Urinalysis System	11065004	All versions

Reason for this Urgent Field Safety Notice

Siemens Healthcare Diagnostics has confirmed that the door gas spring which holds the analyzer door open is failing before its specified lifetime. Due to this failure, a door in the open position may intermittently descend slowly until it reaches an approximately horizontal position, then has the potential to close more rapidly. The closing door has the potential to hit operator who is either unaware of the slowly descending door or who chooses to ignore the closing door while performing routine maintenance or replacing consumables on the system at the time of the failure.

Risk to Health

It is possible that the unexpected closing of the door may result in an impact to the arms or hands of an operator, causing pain. Due to the small mass of the door, the amount of force generated by the falling door is unlikely to be sufficient to cause an injury which would require medical attention.

Actions to be Taken by the Customer

- Until the door gas spring has been replaced, it is recommended that anytime the operator has the door open to perform maintenance or replace consumables, a second operator monitors the door to ensure the door does not close due to the gas spring failure.
- Complete and return the Field Correction Effectiveness Check form attached to this letter within 30 days.
- Review this letter with your Medical Director.
- Keep this letter with your User's Guide for reference, as needed, and forward this letter to those who may have received this product.

The analyzer door gas spring of the affected systems will be replaced.

We apologize for the inconvenience this situation may cause. If you have any questions, please contact your Siemens Customer Care Center or your local Siemens technical support representative.

Sincerely yours,

Siemens Healthcare Diagnostics GmbH

i.A. Roland Ertl, MA

i.A. Mag. Thomas Hufnagl