## Varnostno obvestilo Obvestilo o popravku

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#### STORAGE AND STABILITY

##### **Thawed Unopened:**

When thawed and stored unopened at 2 to 8°C, this product will be stable as follows:

- All Analytes: 45 days

Except:

- Beta-2-Microglobulin: 24 days

- Apolipoprotein A-I: 22 days

- **Anti-cyclic Citrullinated Peptide (Anti-CCP)** and Rheumatoid Factor: **10 days**

##### **Instrument Storage (Siemens Atellica, On-Board, 2 to 8°C):**

This product will be stable as follows:

- All analytes: 30 days

Except:

- Beta-2-Microglobulin: 21 days

- Apolipoprotein A-I: 14 days

- **Anti-cyclic Citrullinated Peptide (Anti-CCP)** and Rheumatoid Factor: **10 days**

Do not refreeze this product after opening the tubes.

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## FIELD ACTION RESPONSE FORM

**Bio-Rad Reference:** FAR 2024-01B

**Bio-Rad Product Segment:** Bio-Rad Laboratories Quality Systems (QS)

**Single Registration Number (SRN):** US-MF-000020712

Product Name		InteliQ Immunology Control				
Level	Catalog Number	Lot Number Expiration Date 2025-01-31	Lot Number Expiration Date 2025-03-31	Lot Number Expiration Date 2025-06-30	Lot Number Expiration Date 2026-02-28	UDI NUMBER
Level 1	12009941	85711T	85721T	85731T	85751T	03610522113842
Level 2	12009942	85712T	85722T	85732T	85752T	03610522113859
Level 3	12009943	85713T	85723T	85733T	85753T	03610522113866
MiniPak	12009944	85710T	85720T	85730T	85750T	03610522113873

## CUSTOMER / CHANNEL PARTNER INFORMATION

Account Name:	
Undersigning Manager Name:	
Address:	
Telephone Number / Fax:	
Account Number:	

### STATEMENT:

- No affected product received
- I am aware of the information about the field action concerning the above reference product(s) and have proceeded according to the instructions issued by Bio-Rad.
- For completion by Channel Partners: All customers have been informed about this field action and have proceeded according to the instructions issued by Bio-Rad. Number of customers informed: \_\_\_\_\_

Number of affected products received:		Number of affected products corrected/ destroyed/ returned (as applicable to the Field Action instructions):	
If number of products corrected/ destroyed/ returned is different to the number received, please account for the difference:			

Date: \_\_\_\_\_

Customer / Channel Partner Signature (and Stamp if applicable): \_\_\_\_\_

Please return this form to: [info@diahem.si](mailto:info@diahem.si)

We need this information to ensure that you have received the product notification.

Thank you for your time and cooperation.